
THUASNE


SPORT

fr	Bracelet strapping Thuasne Sport	4
en	Thuasne Sport strapping wrist band	5
de	Strapping Handgelenkbandage Thuasne Sport	6
nl	Thuasne Sport armband met elastische banden	7
it	Bracciale strapping Thuasne Sport	9
es	Brazaletes strapping Thuasne Sport	10
pt	Bracelete de strapping Thuasne Sport	11
da	Armbånd med stropper Thuasne Sport	13
fi	Nauhallinen Thuasne Sport -kynnärvarsituki	14
sv	Thuasne Sport armband med elastiska band	15
el	Περικάρπιο strapping Thuasne Sport	16
cs	Pásková bandáž zápěstí Thuasne Sport	17
pl	Bandażowa orteza bransoletkowa Thuasne Sport	19
lv	Thuasne Sport aproces ortoze	20
lt	Apjuosiamas riešo įtvaras „Thuasne Sport“	21
et	Thuasne Sport rihmaga kinnitatav randmeside	22
sl	Zapestni trak Thuasne Sport	24
sk	Náramok s páskami Thuasne Sport	25
hu	Thuasne Sport pántos csuklórögzítő	26
bg	Накитник с каишки Thuasne Sport	27
ro	Brățară cu chingă Thuasne Sport	29
ru	Ременный браслет Thuasne Sport	30
hr	Narukvica s trakom Thuasne Sport	31
zh	Thuasne Sport 绷带手环	33
ar	Thuasne Sport سوار لف الرسغ	35

1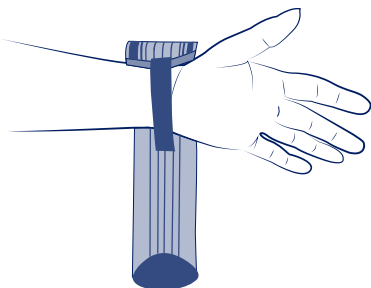

2

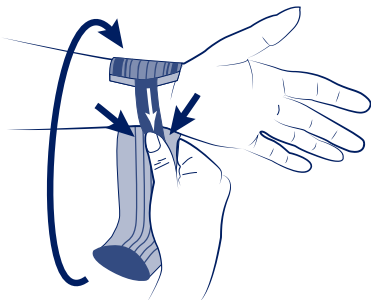fr	tour de poignet	taille unique
en	wrist circumference	universal size
de	Handgelenkumfang	Einheitsgröße
nl	polsomtrek	één maat
it	circonferenza del polso	taglia unica
es	contorno de la muñeca	talla única
pt	perímetro do punho	tamanho único
da	håndleddets omkreds	kun én størrelse
fi	ranteen ympärysmitta	yksi koko
sv	håndledens omkrets	en storlek
el	Περίμετρος καρπού	Ένα μέγεθος
cs	obvod zápěstí	univerzální velikost
pl	obwód nadgarstka	rozmiar uniwersalny
lv	plaukstas locītavas apkārtmērs	universāls izmērs
lt	riešo apimtis	vienas dydis
et	Randme ümbermõõt	Üks suurus
sl	obseg zapestja	enotna velikost
sk	obvod zápästia	univerzálna veľkosť
hu	csukló kerülete	egy méret
bg	обиколка на китката	един размер
ro	circumferința încheieturii	mărire universală
ru	Обхват запястья	Единый размер
hr	obujam ručnog zgloba	univerzalna veličina
zh	腕围	均码
ar	محيط الرسغ	مقاس واحد

BRACELET STRAPPING THUASNE SPORT

Description/Destination

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en

THUASNE SPORT STRAPPING WRIST BAND

Description/purpose

This device is an elastic brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - Elastane - Polyester - Latex - Polyurethane.

Properties/Mode of action

Adjustable fastening system.

Bilateral product.

Indications

Wrist tendinopathies.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Consult a healthcare professional:

- In case of doubt before use,
- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Place the wrist through the positioning belt ❶.

Wrap the strap around the wrist to be reinforced and adjust the compression using the hook-and-loop fastening ❷.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30 °C. Close the self-fastening tab before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2034101 (2022-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186