
THUASNE

SPORT

fr	Genouillère rotulienne renforcée Thuasne Sport	4
en	Thuasne Sport reinforced patellar knee brace	5
de	Verstärkte Thuasne Sport-Kniebandage für die Kniescheibe	7
nl	Thuasne Sport verstevigde kniebrace voor de knieschijf	8
it	Ginocchiera rotulea rinforzata Thuasne Sport	10
es	Rodillera rotuliana reforzada Thuasne Sport	11
pt	Joelheira rotuliana reforçada Thuasne Sport	13
da	Forstærket knæbind til knæskallen Thuasne Sport ...	14
fi	Vahvistettu Thuasne Sport -polvilumpiotuki	16
sv	Förstärkt knäskålsstöd Thuasne Sport	17
el	Ενισχυμένη επιγονατίδα Thuasne Sport	19
cs	Zpevněná česková ortéza Thuasne Sport	20
pl	Orteza kolanowa do rzepki, wzmocniona Thuasne Sport	22
lv	Ceļgala saišu ortoze ar pastiprinājumu Thuasne Sport	23
lt	Sutvirtintas kelio girnelės įtvaras „Thuasne Sport“ ...	25
et	Thuasne Sport põlveketra toetav tugevdatud põlvekaitse	26
sl	Ojačena kolenska opornica za pogačico Thuasne Sport	27
sk	Zosilnená kolenná ortéza Thuasne Sport	29
hu	Thuasne Sport megerősített patella térdrögzítő	30
bg	Подсилена пателарна наколенка Thuasne Sport ...	32
ro	Genunchieră de susținere rotuliană ranforsată Thuasne Sport	33
ru	Усиленный наколенник для поддержки коленной чашечки Thuasne Sport	35
hr	Pojačani steznik za patelu Thuasne Sport	37
zh	Thuasne Sport 加强膝盖骨护膝	38
ar	Thuasne Sport دعم رضفة الركبة المُعزَّز	41

	S	M	L	XL	XXL
cm	32 - 34	35 - 37	38 - 41	42 - 45	46 - 49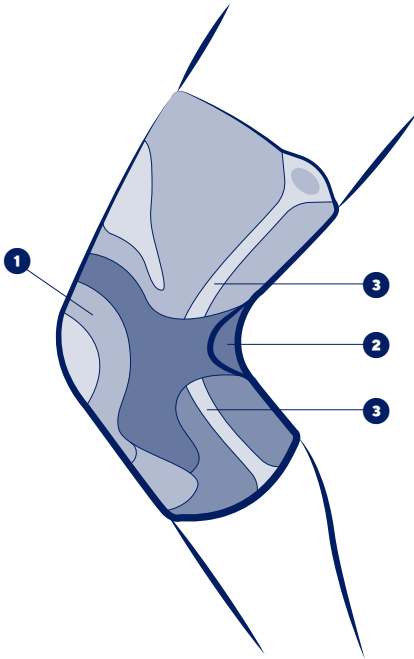

fr	Circonférence de genou en légère flexion
en	Knee circumference in slight flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooidie knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do Joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Celgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenktu kelio apimtis
et	Kergelt pinguldatud põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
ru	Обхват колена в состоянии небольшого сгибания
hr	Obuhvaća koljeno u blagoj fleksiji
zh	轻度弯曲的膝关节周长
ar	محيط الركبة في حالة النسي الخفيف

ff

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en

THUASNE SPORT REINFORCED PATELLAR KNEE BRACE

Description/purpose

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product

Available in 5 sizes.

The device is composed of:

- a patellar insert ❶.
- an anatomically-shaped knitting integrating a specific knit at the back of the knee ❷.
- flexible lateral stays ❸.

Composition

Textile components: polyamide - polyester - polyurethane - elastan - silicone - ethylene vinyl acetate

Rigid components: steel - Polyethylene

Properties/Mode of action

An anatomically-shaped knitting for even compression and proprioception.

The knee support is maintained on the leg thanks to flexible side stays and anti-slip system.

Indications

Anterior knee or patella pain.

Mild patellar instability.

Patella tendinopathy (including jumper's knee syndrome).

Knee pain and/or swelling management.
Sport activities resumption.

Contraindications

Do not use in the event of known allergy to any of the components.
Do not apply the product in direct contact with broken skin.
History of venous or lymphatic disorders.
Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.
Do not use the product if it is damaged.
Choose the appropriate size to fit the patient, referring to the size chart.
For hygiene and performance reasons, do not re-use the product for another patient.
Consult a healthcare professional in case of uncertain diagnosis.
In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.
In the event of any modification in the product's performance, remove it and consult a healthcare professional.
In case of doubt before any sports activity, consult a healthcare professional.
Do not wear the product in a medical imaging machine.
Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.
Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Pull the product on (the logo indicates the top and front of the product).

Make sure that your knee cap is positioned in the centre of the patella ring.

Care

Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2035401 (2022-08)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186