# THUASNE

## ImmoVest

- fr** Gilet d'immobilisation d'épaule  
**en** Shoulder immobilisation vest  
**de** Bandage zur Ruhigstellung der Schulter  
**nl** Schouderorthese voor immobilisatie  
**it** Gilè di immobilizzazione della spalla  
**es** Chaleco para inmovilización del hombro  
**pt** Colete de imobilização do ombro  
**da** Vest til skulderimmobilisering  
**fi** Olkapään immobilisointiliivi  
**sv** Väst för att immobilisera axeln  
**el** Γλέκο ακινητοποίησης ώμου  
**cs** Imobilizační ramenní závěs  
**pl** Ortez do unieruchamienia barku  
**lv** Pleca imobilizācijas veste  
**lt** Peties imobilizavimo liemenė  
**et** Õlaliigest fikseeriv ortoos  
**sl** Steznik za imobilizacijo rame  
**sk** Vesta na znehybnenie ramena  
**hu** Vállrögzítő mellény  
**bg** Жилетка за обездвижване на рамото  
**ro** Orteză „tip vestă” de imobilizare a umărului  
**ru** Жилет для обездвиживания плеча  
**hr** Prsluk za imobilizaciju ramena  
**zh** 肩部固定背心  
**ar** صدرية لتثبيت الكتف

|   |             |
|---|-------------|
| 1 | < 85 cm     |
| 2 | 86 - 95 cm  |
| 3 | 96 - 105 cm |
| 4 | > 105 cm    |

© Thuasne - 6167701 (2022-04)  
© Studio Caterin

**THUASNE**  
120, rue Marius AUFAN  
92300 Levallois-Perret  
France  
[www.thuasne.com](http://www.thuasne.com)

**UK  
CA**

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## SHOULDER IMMOBILISATION VEST

**Description/Intended users**

This device is a shoulder sling allowing the immobilisation of the shoulder and/or the elbow in internal rotation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

**Composition**

Polyester - polyurethane - polyamide - cotton.

**Properties/Mode of action**

Shoulder immobilisation vest allowing for shoulder and/or elbow immobilisation in internal rotation.

Anatomical, wraparound design, covering the shoulder and elbow for reinforced protection.

Comfortable foam structure.

Removable strap for reinforced immobilisation.

Bilateral model.

**Indications**

Shoulder and/or elbow injuries or inflammatory conditions.

Conservative treatment for non-surgical proximal humeral fractures.

Shoulder dislocation.

Non surgical fractures of the clavícula and/or scapula.

Post-operative immobilisation (such as after rotator cuff surgery or dislocation reduction...).

Shoulder subluxation or dislocation prevention (joint instability).

Support for paralytic upper limb.

**Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in certain post-operative conditions (multiple fractures, pin insertion, open fractures, etc.).

Do not use in the event of complex or surgical shoulder fractures, humeral fractures or rib cage fractures.

**Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product when driving a vehicle.

Prolonged immobilisation can sometimes cause muscle weakness. If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

It is recommended to adequately tighten the device to achieve immobilisation without restricting blood circulation.

Wear the device above thin clothing.

**Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

**Instructions for use/Application**

The help of another person is required to put on this product.

**Preparing the device:**

Unfold the device and open the arm sling ☺

Depending on the shoulder to immobilise, reverse the direction of the device: remove the arm sling ☺ and place it on the opposite panel:

- Right shoulder/elbow to immobilize. ①

- Left shoulder/elbow to immobilize. ②

To facilitate application, fold the self-fastening closures of the sleeve, the abdominal and dorsal panels onto themselves.

The removable securing strap ☺ is recommended for a better immobilisation.

Pre-position the removable securing strap ☺ :

- one edge on the dorsal panel ☺ on the indicated area ① (see back view ☺)

- one edge on the top of the sleeve ☺ (see front view ☺).

The strap will be adjusted when closing.

**Fitting the device:**

① Slide the arm needing immobilisation along the sleeve ☺.

② ③ Place the shoulder pad ② onto the shoulder to immobilised.

Unfold the self-fastening closure ☺ of the dorsal panel ☺, attach it onto the front panel ☺. If needed, adjust the length of the abdominal panel ☺ by folding it onto itself.

④ If needed, adjust the closure strap ④ pre-positioned on the side of the immobilized shoulder.

⑤ Place the arm needing immobilisation against the abdomen, fold and close the arm sling ☺ onto the chest.

⑥ Unfold the self-fastening closure and attach the end of the sleeve ☺ under the arm sling ☺.

⑦ ⑧ Reposition and tighten the removable securing strap ☺ for a stronger immobilisation.

To remove the device, detach the removable securing strap, the sleeve, then the arm sling and detach the abdominal and dorsal panels.

**Care**

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). The device has been tested over 20 wash cycles. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

**Storage**

Store at room temperature, preferably in the original packaging.

**Disposal**

Dispose of in accordance with local regulations.

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